

Procedure for Saving Images for Use in Presentations

Following the steps below saves patient images as standard computer image files for non-diagnostic use. Please note that:

- The images are saved in formats that may include lossy compression. They must **not** be used for reading.
- In order to comply with privacy requirements, patient information must be removed from the images after they are saved by using programs such as Windows Paint.


These instructions apply only to Windows XP versions of the Dilon 6800.

To Save an Image to a Flash Drive

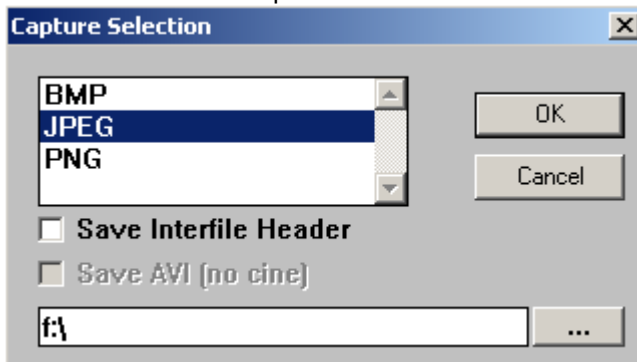
Flash drives are also known as thumb or jump drives.

1. Insert the flash drive into the USB port on the front of the Dilon 6800 computer. A window displaying the contents of the drive may be displayed. If so, close it by clicking on the X in the upper right corner of that window.






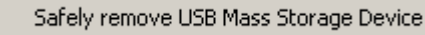
2. From the Dilon software, click on the  button to enter Static Image Processing.
3. Display the desired Static Image Report as done normally for processing or printing. If necessary refer to the appropriate section of the user manual for details.

4. Click on the Screen Capture  button. The window below will be displayed:



5. Click on JPEG if it is not selected.
6. Make sure that "Save Interfile Header" is not checked. If it is, click on the check mark to remove it.
7. Erase any text in the lower box and type in "f:" without the quotation marks.
8. Click on OK.



9. Click on the Exit Door .
10. Save any additional images by following steps 2 through 8. A one gigabyte flash drive has room for thousands of saved images.
11. Click on the Minimize button  on the upper right corner of the Dilon window.
12. Move the cursor to the lower right corner of the screen and click on the Safely Remove Hardware button  on taskbar.
13. Click on the  button when it is displayed.
14. Remove the flash drive when directed.